LifeSource™
Your source for a lifetime of health

Cuff

for Digital Blood Pressure Monitors

Standard Cuff

EasyCuff™ Feature

AND
A&D Medical

Trilingual Instruction Guide

LifeSource™

Your source for a lifetime of health

Congratulations. You have purchased a LifeSource state-of-the-art Blood Pressure Cuff. Our cuffs are made with high quality material to provide comfort and convenience during a measurement. This cuff was designed specifically to be used with LifeSource and A&D Medical Blood Pressure Monitors.

Physicians agree that daily self-monitoring of blood pressure is an important step in maintaining cardiovascular health and preventing the serious consequences associated with undetected and untreated hypertension.

A&D Medical has been manufacturing high quality healthcare products for more than 20 years. We are committed to providing you and your family with monitoring devices specifically designed for high accuracy and ease of use.

TABLE OF CONTENTS

SELECTING THE CORRECT CUFF SIZE . . . E-2

BEFORE YOU START E-3

CONNECTING YOUR CUFF E-4

PLACING YOUR CUFF ON PROPERLY . . . E-5

CONTACT INFORMATION E-6

FRENCH INSTRUCTIONS F-1

SPANISH INSTRUCTIONS. S-1


SELECTING THE CORRECT CUFF SIZE

Using the correct cuff size is important for an accurate reading. A cuff that is too large will produce a reading that is lower than the correct blood pressure; a cuff that is too small will produce a reading that is higher than the correct blood pressure. With your arm hanging at the side of your body, measure the circumference of your upper arm at the midpoint between the shoulder and elbow.

ARM SIZE	RECOMMENDED CUFF SIZE	REPLACEMENT CUFF MODEL#
5.1 - 7.9" (13 - 20 cm)	Small Cuff	UA-279
7.5 - 12.2" (19 - 31 cm)	Medium Cuff	UA-280
7.5 - 14.2" (19 - 36 cm)	Medium EasyCuff™ Feature	UA-380
11.8 - 17.7" (30-45 cm)	Large Cuff	UA-281

■ Cuff Size Indicator On The Cuff

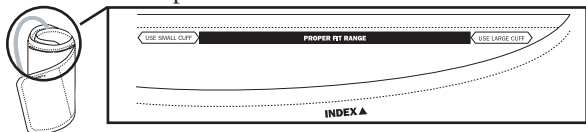Most of our cuffs have an indicator which tells you whether or not you are using the correct cuff size. Place the cuff on your arm (see Pg. E-5 to learn how to put the cuff on correctly) and if the Index Mark ▲ points within the Proper Fit Range, you are using the correct cuff. If the Index points outside of the Proper Fit Range, contact A&D Medical at 1-888-726-9966 (in Canada - 1-800-463-5414) to purchase a cuff replacement.

BEFORE YOU START

For most models of blood pressure monitors, an air connector plug is required to attach the cuff tube to the monitor. Air connector plugs have been provided to accommodate various *LifeSource* and A&D Medical Digital Blood Pressure Monitor designs. Identify the model of your blood pressure monitor and select the corresponding air connector plug from the following table.


MODEL	CONNECTOR
UA-702	T-Connector 
All Other Models Either connector can be used	L-Connector 
	I-Connector 
UA-701, UA-711	No Connector required

CONNECTING YOUR CUFF


Once you have selected the correct air connector plug, refer to the drawings below and complete the following steps:

1. Insert the cuff tube end of the air connector plug into the cuff tube.
2. Insert the monitor end of the air connector plug into the digital blood pressure monitor air socket. You will hear it snap into the monitor.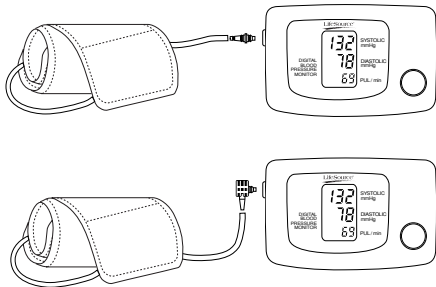

NOTE: For UA-702 models, insert the bulb tube end of the air connector plug into the bulb tube. The bulb and cuff tube ends of the T-Connector plug are interchangeable.

3. When these connections have been made, your blood pressure monitor is ready for operation.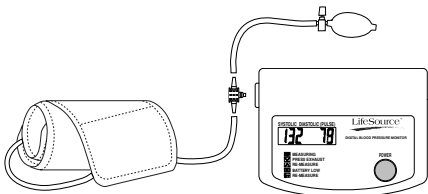

NOTE: Refer to the Instruction Manual that was included with your monitor for operation information


PLACING YOUR CUFF ON PROPERLY

Before you place the cuff on your arm:

- ☞ Relax for about 5 to 10 minutes before measurement.
- ☞ Remove constricting clothing and place cuff on bare arm.
- ☞ Unless your physician recommends otherwise, use left arm to measure pressure.

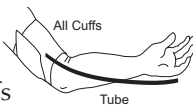Now you are ready. Follow these simple steps:

1. Sit comfortably with your left arm resting on a flat surface so that the center of your upper arm is at the same height as your heart.
2. Lay left arm on the table with palm up. (For cuffs with a metal D-Ring, thread cuff end through metal loop, smooth side against arm). Position the tube off-center toward the inner side of arm in-line with the little finger.
3. Fasten the cuff securely. (To fasten cuffs with a metal D-Ring, pull the end of the cuff to tighten it and fold back the extra material). The cuff should be snug but not too tight. You should be able to insert two fingers between the cuff and your arm.
4. Refer to the instruction manual that was included with your monitor for information on how to use your blood pressure monitor.

CONTACT INFORMATION

Our products are designed and manufactured using the latest scientific and technological methods, and offer accurate, easy to use, home monitoring and treatment options. Our full product line includes:

- **Aneroid Blood Pressure Kits**
 - **Blood Pressure Cuffs**
- **Digital Blood Pressure Monitors**
 - **Digital Thermometers**
- **Heart Rate Monitors**
 - **Nebulizers**
- **Personal Health Scales**
 - **Stethoscopes**

This LifeSource blood pressure product is covered by a Lifetime Warranty.

**Visit our web site at www.LifeSourceOnline.com
for warranty registration.**

For more information regarding use, care or servicing of your blood pressure product, contact:

A&D Medical

A division of A&D Engineering, Inc.

1555 McCandless Drive

Milpitas, CA 95035

LifeSource Health Line (Toll-Free): 1-888-726-9966

www.LifeSourceOnline.com

For Canada Residents, please contact:

Auto Control Medical

206A boul. Brunswick

Pointe-Claire, Quebec

H9R 5P9 Canada

AutoControl (Toll-Free): 1-800-463-5414

www.autocontrol.com

IMPORTANT!

**If You Need Assistance with the
Set-Up or Operation**

We Can Help!

**Please call us FIRST before contacting
your retailer at**

LifeSource Health Line

1-888-726-9966 - Toll Free

1-800-463-5414 - In Canada

A specially trained representative will assist you

A division of A&D Engineering, Inc.
1555 McCandless Drive
Milpitas, CA 95035
www.LifeSourceOnline.com

LifeSource and EasyCuff are trademarks of A&D Medical.
© 2003 A&D Medical. All rights reserved.
Specifications subject to change without notice.

I-MAN:280 Printed in China 4/03

LifeSource™
Your source for a lifetime of health

Brassard

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Brassard standard

Brassard-confort

AND
A&D Medical

Manuel d'instructions trilingue

LifeSource™

Your source for a lifetime of health

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