UC-324ANT

Wireless Precision Scale

Instruction Manual

Before use

Precautions.....	1
Welcome.....	3
Scale Components	3
Get Started	4

Operation

Display Symbols	5
Replacing the Batteries	5

References

Before Requesting Service.....	6
Maintaining the Scale	6
Specifications	6
Other Information.....	7

Precautions

We strongly recommend that you read this instruction manual carefully prior to using the scale for the first time.

Installation

- The scale is designed for indoor use.
- Keep the temperature and the humidity of the installation site within the specified operating temperature range / humidity.
- Place the scale on a solid and level floor.
- Do not place the scale on a soft, wet or slippery floor.

Precautions for use

- Do not use the scale if its glass is cracked or broken.
- Do not step onto your scale with wet feet. Make certain that feet are dry to prevent slipping or serious injury.
- Gently step onto the center portion of your scale. Do not jump on the scale.
- Do not disassemble or modify the scale. Prevent foreign matter (staples, clips, water) from invading the scale.
- Do not place the scale in areas where it can be tripped over or kicked.
- Place the scale where it is not exposed to direct sunlight.
- Place the scale where it is not exposed to strong external noises and electromagnetic waves.
- Place the scale where the radio signal between the scale and receiver won't be significantly impeded by metal or walls.
- Do not place the scale where there are corrosive, flammable, or explosive substances.
- The maximum weighing capacity of the scale is 330 lbs (150 kg). Do not place anything on the scale that is beyond the weighing capacity.
- Do not attempt to service or repair the scale.


Welcome

Congratulations on purchasing a state-of-the-art A&D Medical UC-324ANT Wireless Precision Scale—one of the most technologically advanced weight scales available today! Designed for ease of use and accuracy, the UC-324ANT will effortlessly integrate into your daily wellness regimen.

A&D Medical has been manufacturing high quality healthcare and wellness products for over 20 years. We are committed to providing you and your family with monitoring devices that provide the utmost in accuracy and convenience.

Scale Components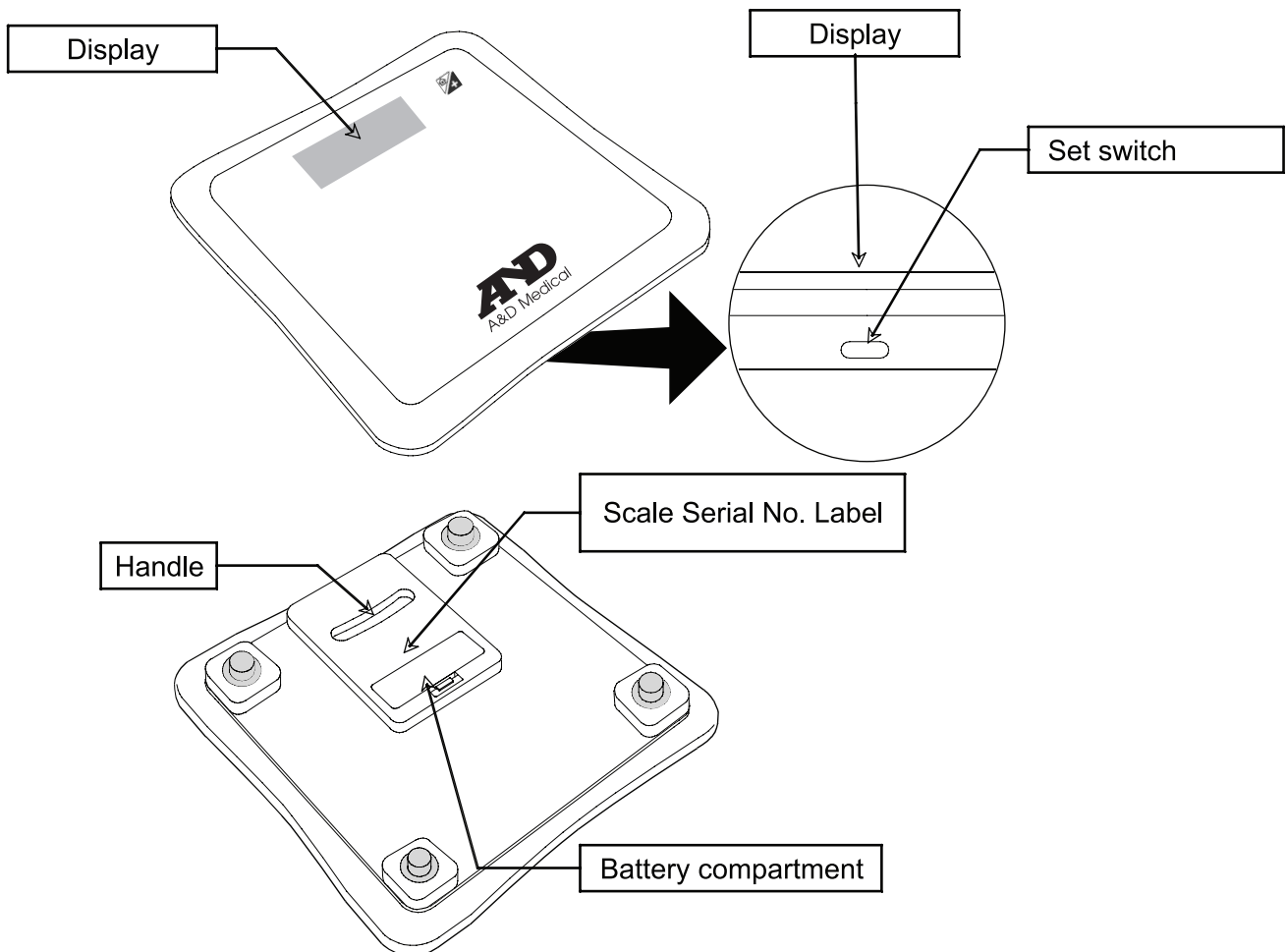


Get Started

Important note – This product is designed to work with different ANT wireless solutions. Please refer to the ANT receiver for more detailed instructions on linking the scale to the receiver

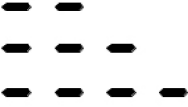
- (a) Insert batteries and immediately place the scale within 1 foot of an active receiver and hold in place for 30 seconds to link the scale to the receiver. If the link fails, remove the batteries and repeat this step.
- (b) Place your scale on a solid, level floor. While the display is off, you may change between pounds and kilograms with the Set Switch. After changing, wait 5 seconds for the scale to return to standby mode.
- (c) Tap the scale with your foot to activate. Wait until the display reads 0.0 lb (0.0 kg). The scale will automatically turn off after 45 seconds if no weight is placed on the scale.
- (d) Step on the scale gently and stand still during measurement. Your weight is displayed after the O symbol appears. The scale will turn off automatically after 15 seconds.
- (e) Step off the scale. The scale automatically transmits your weight wirelessly if your scale is within range of an active radio receiver. If the scale is not in range, the data will be stored in memory until the scale is activated in range. The UC-324ANT can store up to 50 measurements.

Important note – The range of the UC-324ANT may be affected by walls, materials around or between the UC-324ANT and receiver, and other environmental factors.

Important note – If the scale does not read a stable weight within 2 minutes, it will turn off automatically. The scale can be manually turned off by pressing the Set Switch.

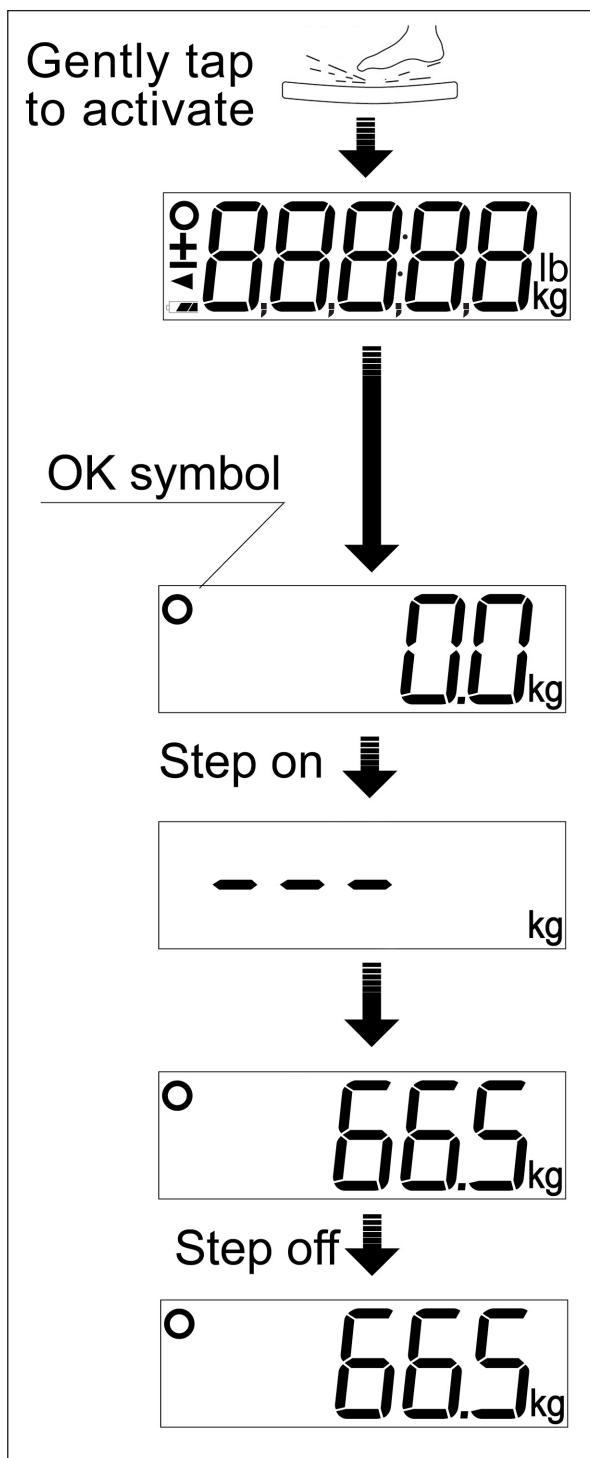Display Symbols

Symbols	Description	Operation
	This symbol appears when the measurement display is stable.	The measurement is complete, you may step off.
	The progress bar displays when the scale is finding a stable measurement. Up to five dashes will appear before a stable measurement is reached.	Continue standing still on the scale until a stable measurement is displayed.
	The battery capacity is very low.	Insert new batteries.


Replacing the Batteries

Caution

- Insert the battery with the positive (+) and negative (-) terminals properly aligned with the corresponding symbols in the battery compartment.
- Replace all four batteries with new ones, when  symbol appears.
- Do not mix used and new batteries. It may result in damage to the scale or shortened battery life.
- When the batteries are depleted, nothing is displayed even if the scale is tapped to activate.
- Remove the batteries from the battery compartment when the scale is not to be used for a long time. They may leak and damage the scale.

Steps for Replacing Batteries

1. Remove the battery compartment cover on the underside of the scale by gently pushing the hook in the direction indicated by the arrow and lifting the cover backward.
2. Install four new batteries with the positive (+) and negative (-) terminals properly aligned with the corresponding symbols in the battery compartment.
3. Replace the cover by sliding it into the compartment and gently pressing into place.

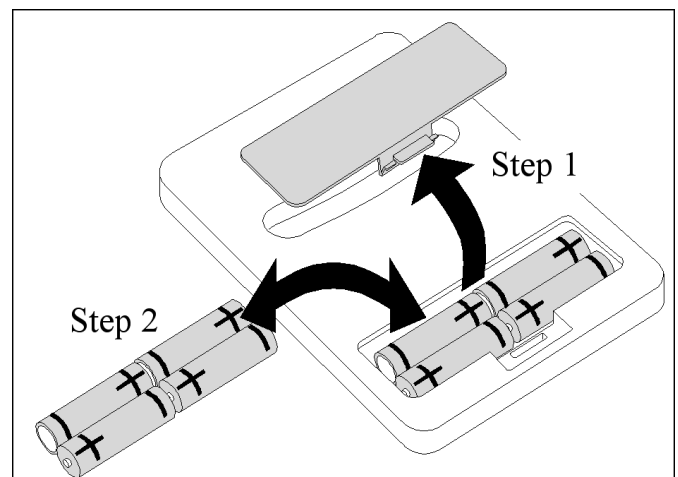Before Requesting Service

Problem	Recommended Action
Nothing appears on the display, even when knocking the scale or pressing the switches.	<ul style="list-style-type: none"> ■ Reinstall the batteries with negative and positive terminals matching those indicated on the battery compartment. ■ Replace all batteries with new ones.
E is displayed.	<ul style="list-style-type: none"> ■ Is anything touching the scale? ■ Is there anything under the scale? ■ Remove and reinstall the batteries. Use new ones.
8888 is displayed and the scale is turned off.	
There is a large difference between actual weight and measured weight	<ul style="list-style-type: none"> ■ Place the scale on a solid, firm, and level surface. ■ Did you allow the scale to zero before applying weight? ■ Is there anything under the scale?
L is displayed.	<ul style="list-style-type: none"> ■ The weight measurement is below the minimum of -22.0lb (-10.0kg). Allow the scale to automatically switch off, then measure again. Be sure the scale is at 0.0lb (0.0kg) before measuring.
H is displayed.	<ul style="list-style-type: none"> ■ The weight measurement is above the maximum of 330.0lb (150.0kg).

Note

If the actions described above do not solve the problem, please contact your dealer.

Do not attempt to open or repair this product, as any attempt to do so will make your warranty invalid.

Maintaining the Scale

- Clean the scale with a soft dry cloth.
- Never use water, detergent, alcohol, benzene, thinner or other harsh chemicals to clean the scale.
- Remove the batteries if the scale is not going to be used for a long time.

Measuring method	Electric resistance
Digital display	Liquid crystal display
Maximum capacity (MAX)	330 lbs / 150 kg
Minimum display (d)	0.2 lbs / 0.1 kg
Power source	Four type AA batteries (1.5 V)
Dimensions	13.8 (D) x 13.8 (W) x 1.5 (H) in / 350 (D) x 350 (W) x 39 (H) mm
Weight	Approximately 4.8 lbs (2.2 kg), including the batteries
Accessories	Instruction manual, batteries (AA x 4)
Recommended operating environment	50°F to 104°F (10°C to 40°C) Less than 85% relative humidity
Recommended storage environment	14°F to 140°F (-10°C to 60°C) Less than 95% relative humidity

Note

Specifications are subject to change without prior notice or any obligation on the part of the manufacturer.

Other Information

Our products are designed and manufactured using the latest scientific and technological methods, and offer accurate and easy-to-use home monitoring and treatment options.

Disclaimer:

A&D Medical assumes no responsibility with regard to the use of these products. Every effort has been made to ensure that the information in this manual is accurate. A&D Medical is not responsible for printing or clerical errors.

These devices comply with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference and (2) this device must accept any interference received, including interference that may cause undesired operations. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the devices.

Contact Information:

For more information regarding use, care, or servicing of your UC-324ANT Wireless Precision Scale, contact your dealer.

A&D Company, Limited
1-243 Asahi, Kitamoto-shi
Saitama 364-8585
JAPAN

Health Line (Toll-Free): 1-888-726-9966 Canada: 1-800-461-0991
www.andmedical.com

© 2010 A&D Medical. All rights reserved.

Specifications subject to change without notice.

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