XL-20 Wireless Activity Monitor Users Manual

English | French | Spanish

AND
A&D Medical

What Does This Work With?

Use your ActiLink™ USB Transceiver with these

**Wellness
Connected™** products:

XL-20
Wireless Activity
Monitor

UA-851THW/UA-851TMW/UA-851THX
Wireless Automatic
Blood Pressure
Monitor

UC-324THW/UC-324TMW/UC-324THX
Wireless Precision
Scale



Getting Started

Before plugging the ActiLink™ USB Transceiver into your computer:



1. Go to www.wellnessconnected.com/getstarted
2. Activate your service plan to receive a registration email.
3. Click on the link within your registration email.
4. Enter in your account information.
5. Register each of your devices and install the ActiLink Client software.
6. Once registration is complete, secure your XL-20 to your footwear.
7. Take a short walk. Your XL-20 does not offload for the first time until you capture over 200 steps of continuous data.
8. Afterward, return to your computer to offload your data and view the results on wellnessconnected.com

Finding The Serial Number

Top

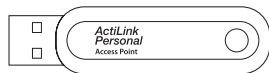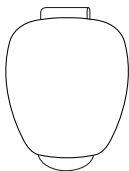Bottom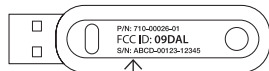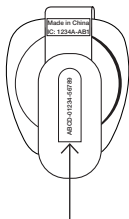

These are the numbers you need to enter at www.wellnessconnected.com during the registration process.

How it Works

The XL-20 utilizes a 3-axis accelerometer to accurately determine distance traveled, calories burned, activity time and total steps. It does not rely on a GPS system, instead providing full coverage and reliability for activities performed indoors or outdoors. The XL-20 will store approximately 10 to 30 days of activity depending on a user's activity levels, and must be worn on footwear (not on a belt or bag).

Recommended Activity Monitor Placement

Good Locations

Bad Locations

Using the ActiLink Software

The ActiLink Software will allow your ActiLink to receive data from the XL-20. The icon in the Windows system tray or Mac dock will indicate the software status.

Good Status:

Everything is working properly and devices can offload!

Warning Status:

The ActiLink Software does not detect an Internet connection and devices cannot be offloaded.

Error Status:

The ActiLink Software does not detect your ActiLink and devices cannot be offloaded.

System Check List

Below are some helpful hints to ensure your Wellness Connected System is working properly:

- ✓ The computer is powered on and not in Sleep, Suspended or Hibernation mode
- ✓ The ActiLink is plugged into the computer's USB port
- ✓ The ActiLink Software icon is blue and visible in your Windows system tray or Mac dock
- ✓ If you still do not see your data on WellnessConnected.com, try taking another short walk of 200 steps or more without stopping, return to your computer and click "refresh page"
- ✓ The XL-20 will offload data when it is "awake" and in range of the ActiLink – the XL-20 is awake when it is on your foot and you are moving, or if you tap it on a hard surface


To check the status of the XL-20, click on the ActiLink Software icon and locate your XL-20 in the Registered Devices list. There are 3 status levels:

- Sleeping** – the XL-20 is not detecting activity and has gone to sleep to save battery life
- Awake** – the XL-20 is detecting activity, but is not offloading data
- Syncing** – The XL-20 is actively offloading data through the ActiLink Software to Wellness Connected

Replacing the Battery


In the Devices Tab on wellnessconnected.com you can check the battery level of your XL-20. If the XL-20 icon is red, then the battery needs to be replaced. Battery life will differ dramatically based on your usage of the XL-20.

1. Using a small Phillips screwdriver, remove the four screws on the bottom of the XL-20.

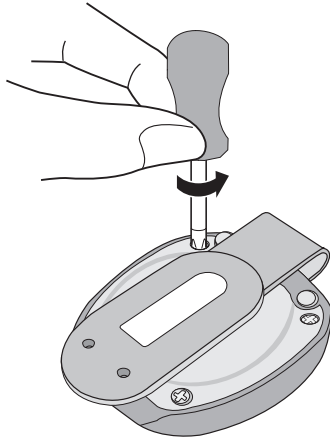Replacing the Battery

2. Separate the two halves by pulling the clip away from the upper housing.
3. Using a flathead screwdriver, lift up the old battery and discard following local battery disposal regulations.
4. Replace with a new CR2032 coin cell battery, positive (+) facing up.
5. Reposition the back cover.
6. Replace the screws so they are tight, but do not over-tighten to the point of stripping the threads.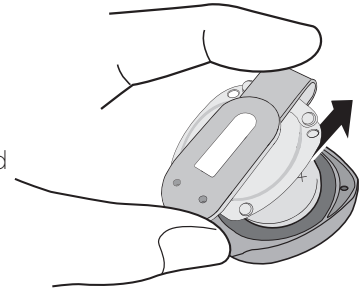

Warranty

This XL-20 Wireless Activity Monitor is covered by a 1 year warranty.

A&D Medical (the "Company") warrants to the first retail purchaser that this A&D Medical XL-20 Wireless Activity Monitor (the "Product") will be free from defects in material and workmanship for one year from the time of purchase. To obtain warranty service for such defects, return the Product, freight and insurance prepaid, together with satisfactory proof of date of purchase such as a sales receipt and shipping costs to:

A&D Medical
Attn: Warranty Department
1756 Automation Parkway
San Jose, CA 95131

Include a check for return shipping and insurance. Contact A&D Medical at 1-888-726-9966 for appropriate shipping costs.

At A&D Medical's option, the Company will repair or replace any Product found to be defective and return it to you. (Any replacement product may be new or refurbished.) This warranty will not apply if the defect or malfunction results from: subjecting the Product to any but the specified voltage, modification, alteration or repairs of the Product by persons not authorized by A&D Medical; misuse, abuse, damage during transit or lack of reasonable care with respect to the Product; or failure to follow the written instructions enclosed with the Product.

THE WARRANTY AND REMEDY PROVIDED ABOVE ARE EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IN NO EVENT SHALL A&D MEDICAL BE LIABLE FOR ANY LOSS, INCIDENTAL, OR CONSEQUENTIAL DAMAGES, RESULTING FROM THE USE OR INABILITY TO USE THE PRODUCT, WHETHER RESULTING FROM BREACH OF WARRANTY OR ANY OTHER LEGAL THEORY, EVEN IF A&D MEDICAL HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH INCONVENIENCE, DAMAGE OR LOSS.

Some states do not allow the exclusion of incidental or consequential damage, so that the above exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that may vary from state to state.

A&D Medical assumes no responsibility with regard to the use of this product. Every effort has been made to ensure that the information in this manual is accurate. A&D Medical is not responsible for printing or clerical errors.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference and (2) this device must accept any interference received, including interference that may cause undesired operations. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this device.

Contact Information

For more information regarding use, care or servicing of your XL-20 Wireless Activity Monitor, contact:

The logo for A&D Medical, featuring the letters 'A' and 'D' in a large, bold, black, sans-serif font. The 'A' and 'D' are connected at the top, and the ampersand is positioned between them.

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A&D Medical Attn: Warranty Department 1756 Automation Parkway San Jose, CA 95131

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The logo for A&D Medical, featuring the letters 'A' and 'D' in a bold, black, sans-serif font, with an ampersand between them.

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H-MAN-XL-20 REV. b

Important!

If you need assistance with the set-up or operation, we can help! Please call us **FIRST** before contacting your retailer at A&D Medical Customer Service Line 1-888-726-9966 (Toll Free). A specially trained representative will assist you.

A&D Medical

**A division of A&D Engineering, Inc.
1756 Automation Parkway
San Jose, CA 95131
www.andmedical.com**

© 2011 A&D Medical. All rights reserved.
LifeSource, the LifeSource logo, Wellness Connected, and the Wellness Connected logo are trademarks of A&D Medical registered in the U.S.

THE XL-20 WIRELESS ACTIVITY MONITOR is covered by one or more of U.S. patent numbers 6336365, 6018705, 6052654, 6493652, 6298314, 6560903, 6611789, 6536139, 6876947, 6882955, 7187924 and patents pending.

SPECIFICATIONS SUBJECT TO CHANGE WITHOUT NOTICE.

Printed in China 03/2011

I-MAN-XL-20 REV. b

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Una división de A&D Engineering, Inc.