

Fast. Easy. Accurate.

WELLNESS CONNECTED WIRELESS BLOOD PRESSURE MONITOR UA-851THX

The Wellness Connected™ Wireless Automatic Blood Pressure Monitor helps provide you with a complete record of your blood pressure and pulse rate. Track your daily blood pressure with the Wellness Connected Online Service right on your home computer. Graphically view your blood pressure readings over weeks, months, or years to easily see your trends and keep you moving towards better health! Your blood pressure is measured, uploaded wirelessly to any computer, and automatically charted with a single press of a button.

- **Wireless Connectivity** – Using patented wireless technology your blood pressure readings will be effortlessly transmitted to your online Wellness Connected service account.
- **Cloud Computing** – Your data is stored in the cloud, meaning you can access it from anywhere you need with an internet connection!
- **Pressure Rating Indicator™** – provides a snapshot of the user's blood pressure classification according to the JNC7 guidelines.
- **Irregular Heartbeat Detection** – alerts the user to the presence of an irregular heartbeat while still providing an accurate reading.
- **Internal Memory** – stores the last 51 readings.
- **4 AA Batteries Included.**

A&D Medical blood pressure monitors lead the way in technology and quality – because peoples' lives depend on them. Our commitment to patient-focused thinking has made us the leading primary manufacturer in developing clinically valuable features. A&D Medical blood pressure monitors are fast, easy, accurate and carry a the industry's strongest warranties.

For questions about Wellness Connected products, call us toll-free at 1-888-726-9966 and a specially trained representative will assist you, or visit us at www.WellnessConnected.com.



Go Completely Wireless!



Use with
XL-10*



*sold separately



- Wireless Connectivity
- User-friendly Software
- Pressure Rating Indicator™
- Irregular Heartbeat Detection
- 50+ Memory Recall
- Displays Average Reading
- Trilingual Instructions
- Illuminated Start Button
- Built-in Cuff Holder
- Fast Measurement
- Track Your Blood Pressure and Pulse Rate Online

UA-851THX Wireless Blood Pressure Monitor

To obtain an accurate reading, it is important to use the correct cuff size. The UA-851THX Wireless Blood Pressure Monitor includes a Medium cuff. A cuff that is too large will produce a reading that is lower than the correct blood pressure; a cuff that is too small will produce a reading that is higher than the correct blood pressure. If you need a Large or another Medium cuff, please contact A&D Medical or visit our online store at www.andmedical.com



PART NUMBER	UPC NUMBER	CUFF SIZE
UA-851THX	0-93764-60282-5	Wireless Blood Pressure Monitor

Accessories

PART NUMBER	UPC NUMBER	SPECS
UA-280	0-93764-04607-0	Medium Cuff: 9.4" - 14.2" (24 - 36cm)
UA-281	0-93764-01069-9	Large Cuff: 14.2" - 17.7" (36 - 45cm)
TB:233	0-93764-60235-1	AC Power Adapter

Technical Specifications

Dimensions	5.9" (w) x 4.9" (l) x 6.1" (h) (150mm x 126mm x 156mm)
Weight	12.3 oz. (350g) without batteries
Display	Digital, 20-mm character height Pressure/pulse displayed simultaneously
Pressurization	Automatic, using micropump
Deflation	Active exhaust valve
Type	Oscillometric
Accuracy	Pressure: ± 3 mmHg or $\pm 2\%$, whichever is greater Pulse: $\pm 5\%$
Pressure Range	Pressure: 20 mmHg to 280 mmHg Pulse: 40 pulses to 180 pulses
Power Source	4 type "AA" (1.5 volt) alkaline batteries (included) or 120 V AC Adapter (optional)
Battery Life	Approximately 4 months with one daily measurement
Operating Environment	50°F to 104°F (10°C to 40°C) Less than 85% relative humidity
Storage Environment	14°F to 140°F (-10°C to 60°C) Less than 95% relative humidity

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method within the limits prescribed by the American National Standards Institute (ANSI/AAMI SP10) for electronic sphygmomanometers. Manufactured by an ISO 9000 and ISO 13485 registered company. Engineered in Japan. Made in China.

Packaging and Shipping Specifications

Selling Unit (Box) UA-851THX	W x D x H (inches) 7.0 x 5.0 x 6.6	Wt (lb.) 2.29	Individual Item Cube (cubic feet) 0.13
Unit Case (10 units) UA-851THX	W x D x H (inches) 14.8 x 31.0 x 7.5	Wt (lb.) 26.0	Case Cube (cubic feet) 1.99

Ask your A&D Medical Sales Representative about the entire Wellness Connected family of products.



1756 Automation Parkway
San Jose, CA 95131
1-888-726-9966
www.andmedical.com

Wellness Connected and LifeSource are registered trademarks of A&D Medical. ©2011 A&D Medical. All rights reserved.



Join to get your real-time health and activity data at WellnessConnected.com

Ask your A&D Medical Sales Representative about other A&D Medical & Wellness Connected products.

- Blood Pressure Monitors
- Digital Thermometers
- Personal Scales
- Stethoscopes
- Activity Monitors

For questions about A&D Medical products, call us toll-free at 1-888-726-9966 or visit one of our websites at:
www.andmedical.com
www.WellnessConnected.com
www.LifeSourceOnline.com

LIT-UA-851THX 06/11